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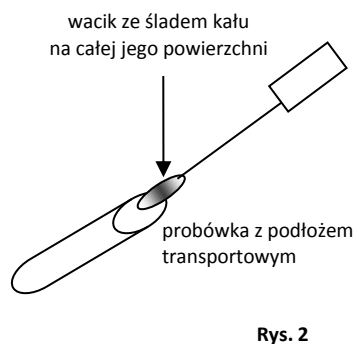
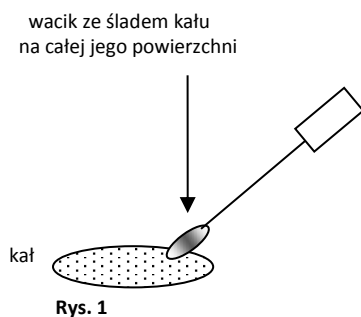
A guide to sampling, storage and transport of the stool samples

A stool sample should be collected using a **cotton swab** from the Transport Medium Kit (available in the Sanitary and Epidemiological Station in Bydgoszcz). The stool sample needs to be collected on 3 consecutive days. If this is not possible, **the interval between the first and the third sample collection cannot be longer than 7 days**. The procedure is consistent with the Laboratory Quality Manual of the Sanitary and Epidemiological Station (pol. WSSE) in Bydgoszcz.

Correct sample collection using cotton swabs:

- Stool should be collected to a clean and dry container.
- A cotton swab should be used to take several smears so that a visible residue of stool is present on over the entire surface the swab (**Figure 1**).
- The swab should be placed inside a test tube and securely closed. The test tube must be clearly labelled with a person's name and surname as well as the date and time of sample collection (**Figure 2**).
- Samples should be stored for no longer than 48 hours after first sample in a refrigerator (2-8°C)

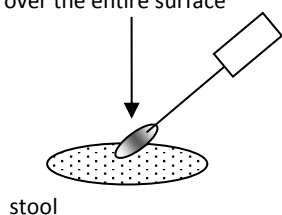
Swabs should be submitted to the Sample Collection and the Results Collection Unit (pol. Punkt Przyjmowania Próbek i Wydawanie Wyników, pol. PPPiWW) in WSSE in Bydgoszcz on Kujawska 4 (Building E) from Monday to Thursday from 7³⁰am till noon and in addition on Thursday from 2pm till 5pm. A clearly filled in form needs to be attached to each submitted sample (the form can be obtained from PPPiWW).

The results may be received up to 4 days after the last sample submission (during weekday), daily from **7³⁰am till 14¹⁵pm**, and in addition on Thursday **from 2pm till 5pm**. A test must be paid (according to the WSSE price list) **upon the first sample submission**. If the payment be due on a Thursday afternoon, an exact amount of money have to be prepared. An invoice or its copy must be shown during receiving of results.

Please note


Do not place a stool sample inside the transport medium (tube). Only the cotton swab (with visible stool residue over the entire surface) should be placed inside the tube. Incorrectly collected specimen will not be analysed!

cotton swab with visible stool residue
over the entire surface

Rys. 1

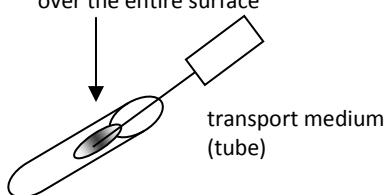cotton swab with visible stool residue
over the entire surface

Rys. 2